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**GB Anti-venom first aid kit**
The ASPIVENIN® mini-pump relieves pain caused by stings or bites of venomous animals. It is a first aid medical device that does not replace the consultation of a physician, which is essential in serious cases. ASPIVENIN® is intended to be used on any person, regardless of age (adult, child, elderly), suffering from bites or stings of insects (mosquitos, bees, wasps) and other venomous animals (spiders, scorpions, weaver fish, stonefish). ASPIVENIN® can be used with or without the help of another person.
**PRINCIPLE:** ASPIVENIN® sucks the venom out painlessly, using instant and permanent vacuum 10 times more powerful than oral suction.
**RECOMMENDED USE:** To ensure proper use, we recommend handling your ASPIVENIN® before using it, consulting the instructions for use. For optimal effectiveness, it is recommended that it be used within 3 minutes of the sting/bite. Depending on the venomous animal, the suggested suction time is as follows: see table 2.
**CONTRAINDICATIONS:** Do not use ASPIVENIN® on the eyelids, mucosa, genitals or ear canal.
**ATTENTION:** ASPIVENIN® does not replace or should not delay the mandatory and immediate consultation of a physician, which is essential in case of a scorpion sting, snake bite, or bite or sting of any other animal in the event of a significant reaction. ASPIVENIN® has removable components (suction cups) that should not be swallowed. Keep this device out of the reach of children. ASPIVENIN® may be damaged in case of fall or impact. Do not drop. In case of allergic reaction, ASPIVENIN® does not prevent the occurrence of angioedema.
**CLEANING, STORAGE and DISPOSAL:** Only the suction cups can be washed with soapy water. Keep the pump away from moisture. ASPIVENIN® must be stored in its box, in a dry place and at room temperature. ASPIVENIN® can be used multiple times if the conditions for use, cleaning and storage described in these instructions for use are respected. Dispose of the device and its packaging in accordance with local or national regulations. Any serious incident in connection with ASPIVENIN® must be reported to DOMES PHARMA (aspivenin@auvexpharma.com) and to the competent authority of the Member State in which the user and/or the patient is located.

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